
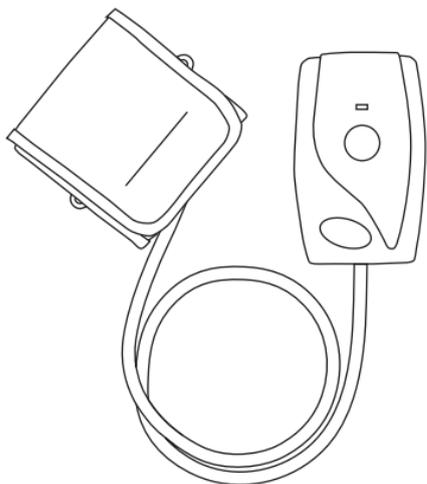
Pulsewave[®] Quick Start Guide

1


Contents

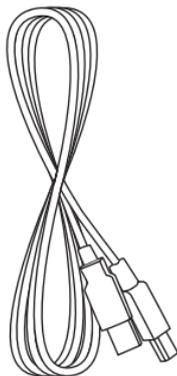Pulsewave® PAD
(Pulse Acquisition Device)
with Inflatable Wrist Cuff

USB cable

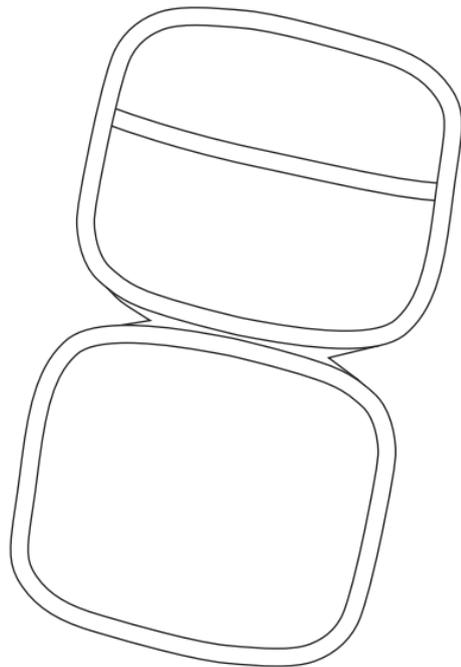
Carrying Pouch

2

Connect

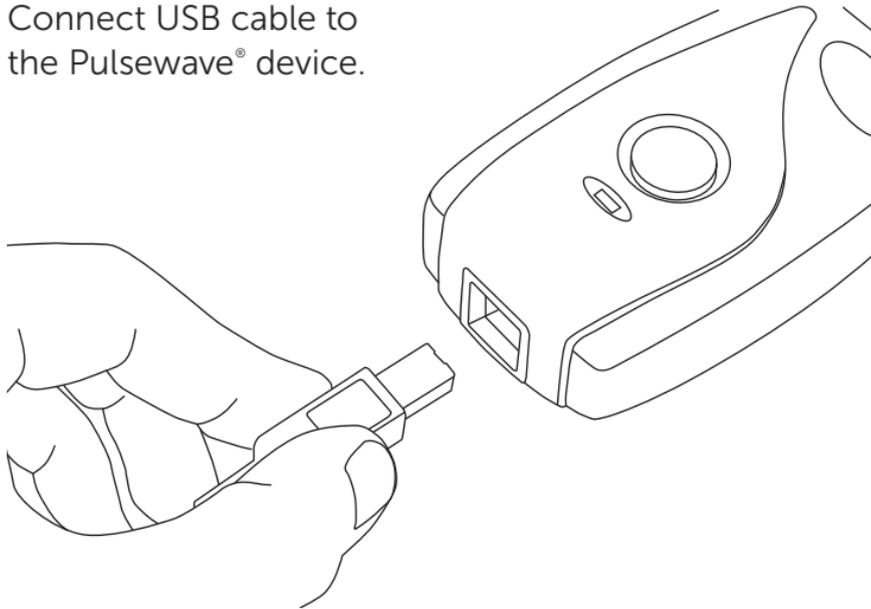
Connect USB cable to the Pulsewave® device.

3

Connect

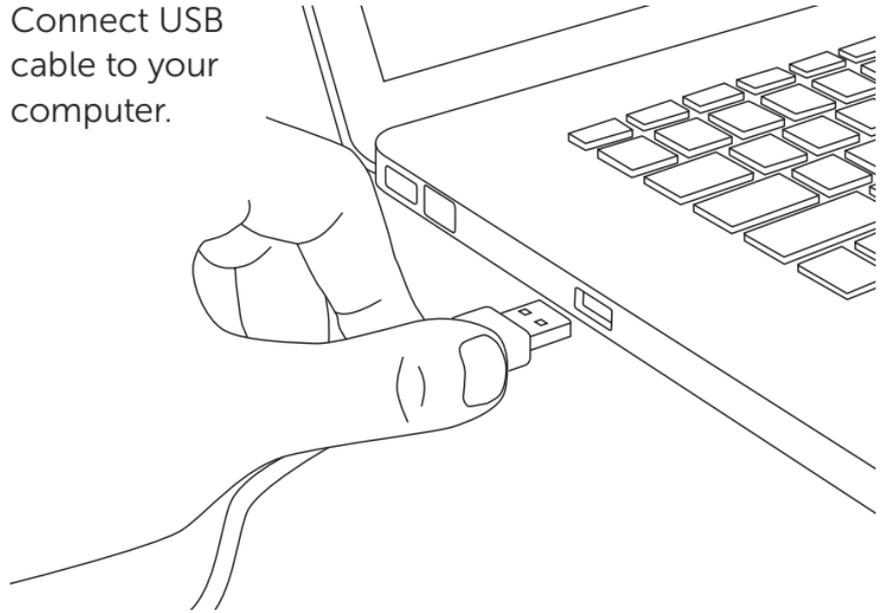Connect USB
cable to your
computer.

4

Register

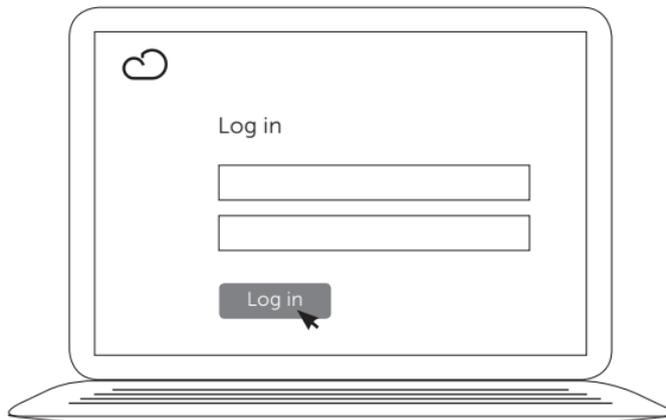
If you have your user name and password, please skip to step 5. Otherwise, please visit **www.pulsewavedx.com** and create your account.

5

Log in


Please visit www.pulsewavedx.com and log in with your user name and password.

6

Attach cuff

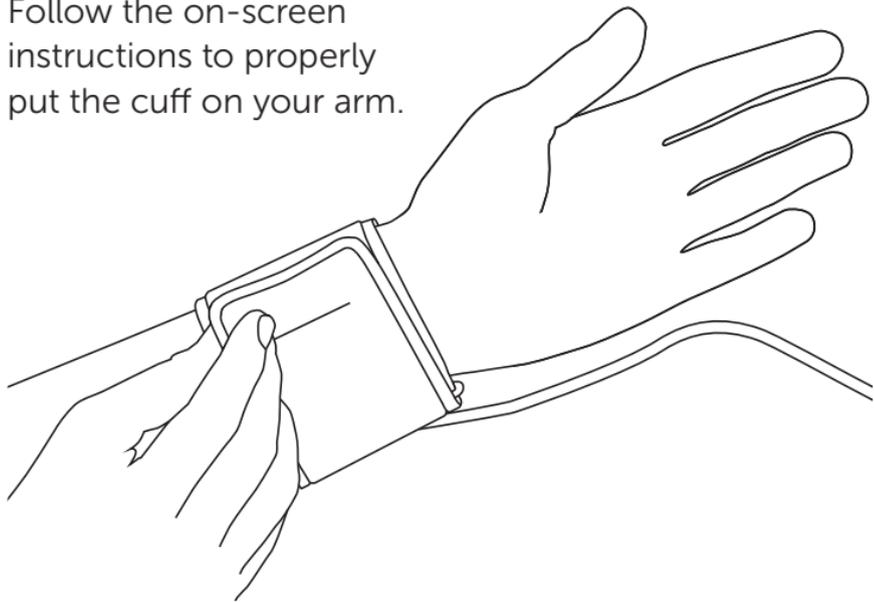Follow the on-screen instructions to properly put the cuff on your arm.

7

Take a reading

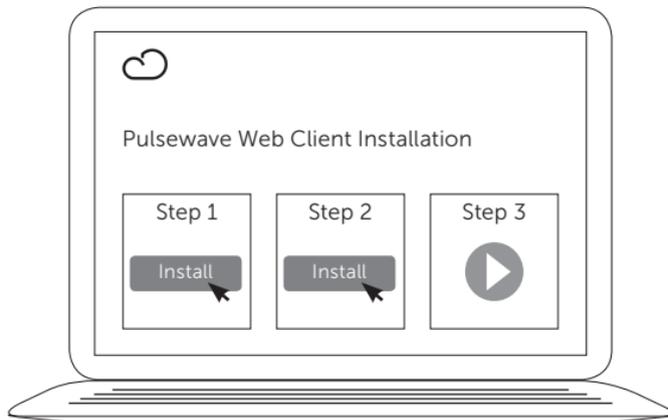
Press the start button to take a reading.

8

Install

If this is your first time you may be prompted to install our plug-in. If prompted, install required software and follow instructions on the screen. When completed, your Pulsewave® will be ready for use.

9

Support

Cloud DX maintains a convenient support web interface that includes an extensive knowledge base, describing all of the functions of Pulsewave®. Please explore the knowledge base here:

<https://support.pulsewavedx.com>

For more information and to receive answers to your questions, please contact Cloud DX Support:

Telephone: **Toll Free: 1-888-543-0944 x207**

Email: **support@clouddx.com**

Hours: **Monday - Friday 9:00 - 17:00 Eastern Time**

Things to know

- ❖ Intended for use only by adults (18 years or older).
- ❖ Not intended to replace regular medical examinations.
- ❖ Do not adjust your medication, diet, or lifestyle without first consulting a physician.
- ❖ Do not use the cuff on skin with cuts, scrapes, open wounds etc.

Conditions that adversely affect use

- ❖ Temperature and relative humidity outside the specified operating range
(refer to Technical specifications at pulsewavedx.com)
- ❖ Mechanical vibrations
- ❖ Incorrect positioning of the cuff
(eg. placed too far up from the wrist, too loose or too tight)
- ❖ Wrist not aligned at heart level
- ❖ Any movement, including talking, during the reading
- ❖ Your physiological condition (such as arrhythmias, and pregnancy)

 Manufactured by:

Cloud DX Inc.

#119 - 151 Charles Street West
Kitchener ON
N2G 1H6
clouddx.com

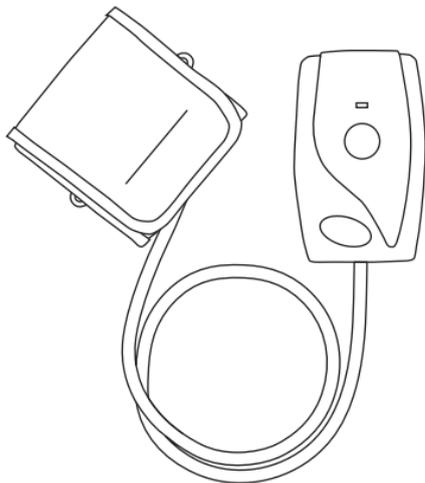
European Representative:
Advena Ltd., Pure Offices
Plato Close, Warwick CV34 6WE UK

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© 2015 Cloud DX Inc